

# ARGEN<sup>®</sup> RESINS

## CURE BOX COMPATIBILITY

Unit	Model	ARGEN MODEL PRO	ARGEN MODEL	ARGEN FLEX	ARGEN GUIDE	ARGEN TRAY
Ackuretta (with Ackuretta)	Curie	5 minutes total, P-16 D-10 BL-ON	10 minutes total, P-16 D-10 BL-ON	10 minutes total, P-16 D-10 BL-ON	5 minutes total, P-16 D-10 BL-ON	10 minutes total, P-16 D-10 BL-ON
Ackuretta (with Ackuretta)	Curie Plus	5 minutes total, P-16 D-10 BL-ON	10 minutes total, P-16 D-10 BL-ON	10 minutes total, P-16 D-10 BL-ON	5 minutes total, P-16 D-10 BL-ON	10 minutes total, P-16 D-10 BL-ON
Asiga	Flash	10 minutes total	Not available	Not available	Not available	Not available
B9Creations	B9 Model Cure	3 minutes per side	Not available	Not available	Not available	Not available
Eurolight (with Flashforge Hunter)	N/A	Not available	Pending	Not available	3.5 minutes per side	Pending
MeccatroniCore (with Asiga <sup>®</sup> )	BB Cure	Not available	10 minutes total	Not available	Not available	50 minutes total
NextDent (3D Systems)	LC-3DPrint Box	Not available	Not available	15 minutes total	30 minutes total	15 minutes total
OtoFlash	G171	250 flashes per side	2000 flashes per side	2000 flashes per side	2000 flashes per side	2000 flashes per side
Dreve	PCU 90	3 minutes in air	Not available	Not available	Not available	Not available
Photopol (with Asiga <sup>®</sup> )	A5408V	Not available	10 minutes total	50 minutes total	40 minutes total	50 minutes total
Phrozen	Cure	Not available	20 minutes per side	40 minutes per side	20 minutes per side	30 minutes per side
Rapid Shape	RS Cure	Not available	5 minutes total upper and lower wavelength, 170 MAX power, 50 mbar vacuum	6 minutes total upper and lower wavelength, 170 MAX power, 50 mbar vacuum	3 minutes total upper and lower wavelength, 170 MAX power, 50 mbar vacuum	6 minutes total upper and lower wavelength, 170 MAX power, 50 mbar vacuum
SHINING 3D	Fab Cure	Not available	Pending	20 minutes per side	20 minutes total	40 minutes total
SprintRay	Pro Cure	Not available	15 minutes per side @ 50C	30 minutes total @ 50C	45 minutes total @ 50C	15 minutes total @ 50C
Uvitron	Intelliray 400W	40 seconds total 10 sec / side x 2 @ 50% power	2 minutes per side	2 minutes per side	150 seconds per side at 50% intensity	2 minutes per side
Wicked (with Asiga <sup>®</sup> )	CureBox	Not available	Argen MODEL Beige: 20 minutes per side @ 60C Argen MODEL Light Grey: 40 minutes per side @ 60C	20 minutes per side @ 60C	5 minutes per side @ 60C	5 minutes per side @ 60C

\*Not a trademark of The Argen Corporation.

The Argen Corporation

8515 Miralani Drive, San Diego CA 92126  
argen.com | customersuccess@argen.com | (800) 255-5524



BD-18938-Ver5-09/24