

Post Curing Values



Resin	(T) Time (min)	(P) Intensity	(D) Duty Cycle	(B) BL
Flex Model Beige/Light Grey	10	16	10	ON
Guide	5	16	10	ON
Tray	7	16	10	ON

* Data provided by Ackuretta

The Argen Corporation

8515 Miralani Drive, San Diego CA 92126
argen.com | customercare@argen.com | (800) 255-5524





Resin	(T) Time (min)	(P) Intensity	(D) Duty Cycle	(B) BL
Temp Crown plus	5	13	8	ON
Model	10	13	10	ON

* Data provided by Ackuretta



Resin	(T) Time (min)	(P) Intensity	(D) Duty Cycle	(B) BL
Guide Model Splint IBT Temp Element Beige/White	10	13	8	ON
Cast Denture	5	13	8	ON
Tray	7	13	8	ON
Gingiva	20	13	8	ON

* Data provided by Ackuretta



Resin	(T) Time (min)	(P) Intensity	(D) Duty Cycle	(B) BL
Crown & Bridge	5	13	8	ON
Denture Base				
Denture Teeth				
Try-In				



Resin	(T) Time (min)	(P) Intensity	(D) Duty Cycle	(B) BL
CROWNTEC	6	10	16	ON

* Data provided by Ackuretta

The Argen Corporation

8515 Miralani Drive, San Diego CA 92126
argen.com | customercare@argen.com | (800) 255-5524



MKT-18959-Ver1-06/22



Resin		(T) Time (min)	(P) Intensity	(D) Duty Cycle	(B) BL
Guide Model GB Model Ivory Model Grey Ortho Model	IBT Splint Temp Match Mock up	10	13	8	ON
Cast Denture		5	13	8	ON
Tray		7	13	8	ON
Gingiva		20	13	8	ON

* Data provided by Ackuretta

The Argen Corporation

8515 Miralani Drive, San Diego CA 92126
argen.com | customercare@argen.com | (800) 255-5524



MKT-18959-Ver1-06/22

Resin	(T) Time (min)	(P) Intensity	(D) Duty Cycle	(B) BL
KeyMask KeyModel Ultra KeyOrtho Model KeyModel KeyOrtho IBT KeySplint Hard KeySplint Soft KeyTray	10	16	10	ON
KeyGuide	5	16	10	ON

* Data provided by Ackuretta

The Argen Corporation

8515 Miralani Drive, San Diego CA 92126
 argen.com | customercare@argen.com | (800) 255-5524





Resin	(T) Time (min)	(P) Intensity	(D) Duty Cycle	(B) BL
Guide Temp Splint IBT Model GB Ortho Model	10	13	8	ON
Cast Denture	5	13	8	ON
Tray	7	13	8	ON
Gingiva	20	13	8	ON


* Data provided by Ackuretta

The Argen Corporation

8515 Miralani Drive, San Diego CA 92126
argen.com | customercare@argen.com | (800) 255-5524



MKT-18959-Ver1-06/22

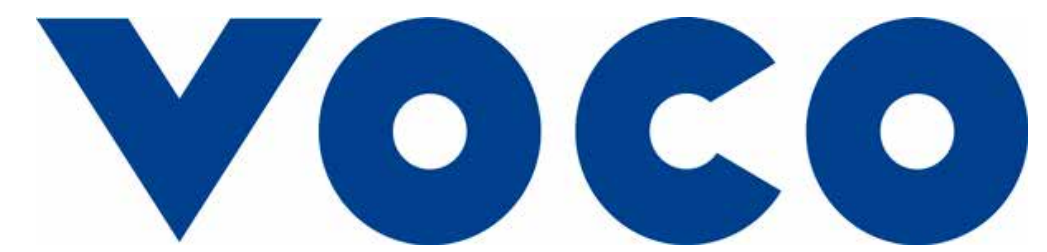

Resin	(T) Time (min)	(P) Intensity	(D) Duty Cycle	(B) BL
Model Beige Model Fast Splint comfort SG Splint Tray Try-In	5	13	8	ON
Cast Dentbase	7	13	8	ON

* Data provided by Ackuretta

The Argen Corporation

8515 Miralani Drive, San Diego CA 92126
argen.com | customercare@argen.com | (800) 255-5524



MKT-18959-Ver1-06/22



Resin	(T) Time (min)	(P) Intensity	(D) Duty Cycle	(B) BL
VeriCast	5	13	8	ON
VeriTray	7	13	8	ON
VeriGuide VeriModel Golden Brown VeriModel Grey VeriModel Ivory VeriModel White	10	13	8	ON

* Data provided by Ackuretta

The Argen Corporation

8515 Miralani Drive, San Diego CA 92126
argen.com | customercare@argen.com | (800) 255-5524



MKT-18959-Ver1-06/22



Resin	(T) Time (min)	(P) Intensity	(D) Duty Cycle	(B) BL
Ocean Blue	5	13	8	ON
Beluga White Orca Black Dolphin Grey	10	13	8	ON



Resin	(T) Time (min)	(P) Intensity	(D) Duty Cycle	(B) BL
Cast&Press	5	13	8	ON
Tray	7	13	8	ON
Gingiva Model Plus UV Grey Model Plus UV Sand Model UV Smoky white	10	13	8	ON

* Data provided by Ackuretta

The Argen Corporation

8515 Miralani Drive, San Diego CA 92126
argen.com | customercare@argen.com | (800) 255-5524



MKT-18959-Ver1-06/22