

ArgenZ Tissue Shades Instructions for Use



ArgenZ Tissue Shades create a pink-shaded foundation to build gingival anatomy. Apply the tissue liquids to zirconia prior to sintering for a natural tissue hue to build upon with your material of choice.

ArgenZ Tissue Shades include pink, light pink, and dark pink.

Device Components: Er_2O_3 , Cr_2O_3

Additional ArgenZ Tissue

Shade Components: Propylene Glycol, H_2SO_4 , De-ionized Water

NOTE: The accessory components listed above are not present in the finished device. The "Device Components" listed above are residual components in the oxidative state of the finished custom device. Safety information for the accessory components in the liquid state are available on the SDS for the accessory.

Warning/Contraindication: Not intended for use in patients who are sensitive or allergic to the chemical components of the device. Please see the Safety Data Sheet (SDS) for detailed device material information.

Precaution: Avoid direct skin or eye contact with these devices. Please see the Safety Data Sheet (SDS) for additional toxicological information.

ArgenZ Tissue Shade Liquid Shading System

Tissue Shade Application

ArgenZ Tissue Shade

ArgenZ Tissue Shades may be applied to any area of green state zirconia, prior to sintering, intended to match natural soft tissue.

- 1) Apply 3 generous coats of ArgenZ Pontic Reducer to any area of tissue that is intended to be colored with ArgenZ Tissue Shades. ArgenZ Pontic Reducer will minimize the absorption of chroma colors during immersion into that area.
- 2) Apply ArgenZ Incisal Effects and any liquid modifier to the tooth portion of the restoration as desired.
- 3) Immerse the green state restoration into the appropriate shade of ArgenZ Liquid Shade for 30 seconds. Blot excess liquid with a clean, dry tissue. Allow to dry for 45 minutes.
- 4) Apply desired shade of ArgenZ Tissue Shade over the tissue surfaces where pontic reducer was previously applied. Apply 3 to 4 even coats. Try keep the modifier approximately .5mm away from tooth surface to minimize a wicking effect into an undesired location.
- 5) Allow to dry 1 hour before placing into sintering furnace. Sinter the restoration following the directions provided by the zirconia manufacturer.

For reduced intensity, ArgenZ Tissue Shades may be diluted with distilled water as desired.

