

ArgenZ HT+ Coloring Dilution Guide

ZirkonZahn Water Based and Origin Live 6.0 Coloring Liquids



ZirkonZahn® Water Based Liquid

Shading Liquid / Deionized Water

A1	100/0
A2	100/0
A3	100/0
A3.5	100/0
A4	90/10

ZirkonZahn® Water Based Liquid

Shading Liquid / Deionized Water

B1	90/10
B2	90/10
B3	100/0
B4	100/0

Use Origin® Live 6.0 Liquid

Shading Liquid / Deionized Water

C1	100/0
C2	100/0
C3	100/0
C4	100/0

ZirkonZahn® Water Based Liquid

Shading Liquid / Deionized Water

D2	90/10
D3	100/0
D4	100/0

Use Origin® Incisal 2.0 Liquid

Use Origin® Incisal 2.5 Liquid

A1, A2, A3	A3.5, A4
B1, B2, B3	B4
C1, C2	C3, C4
D2	D3, D4

NOTE: If a lighter shade is desired a 90/10 ratio will produce 1/2 – 1 shade lighter. Please call Argen for assistance with adjusting your liquid to water ratio.

