

Barcode

1. Lab Info
2. Order Date and Ship Date
3. Case Name
4. Sales Order (D#)
5. Case Information
6. Product Category



Unassembled Dentures

- Includes the printed pink base and printed tooth shade arch.
- See Argen Assembly Instruction for specific light curing and additional materials required.
- Available in the following Pink Base Shades: Original Pink, Dark Pink, Light Pink, Reddish Pink
- Available in the following Tooth Shades: A1, A2, A3, A3.5, B1, B2



Assembled Dentures

- Includes the printed pink base and printed tooth shade arch that has been assembled, surface treated, and UV cured.
- The denture will need to be polished and high shined at the lab.
- Characterization optional.
- Available in the following Pink Base Shades: Original Pink, Dark Pink, Light Pink, Reddish Pink
- Available in the following Tooth Shades: A1, A2, A3, A3.5, B1, B2



Try-In Dentures


- Includes the printed tooth material in a 1-piece, monolithic denture.
- Available only in white.
- May require polishing at the lab.

Digital Dentures | What to Expect



Tools Required (Washing)

1. Personal protective equipment
2. Disposable gloves
3. Tray
4. Paper towels
5. Foil
6. Part removal tool
7. Silicone tongs
8. Labeled wash container
9. Isopropyl alcohol
10. Orbital shaker
11. Lab timer

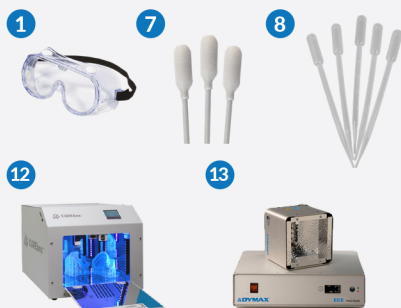

Tools Required (Setting & Surface Prep)

1. Personal protective equipment
2. Disposable gloves
3. Tray
4. Paper towels
5. Foil
6. Bristle brush
7. Swabs
8. Pipettes
9. Cup
10. Hand tools/bur
11. Glass plate
12. Desktop LED UVA cure unit (or gel nail/dental crown curing lamp)
13. Dymax ECE 5000 or Dreve PCU LED UV curing systems



Tools Required (Curing)

1. Personal protective equipment
2. Tray
3. Paper towels
4. Kimwipes
5. Foil
6. 2 pairs of silicone tongs
7. Lab timer
8. Dymax ECE 5000 or Dreve PCU LED UV curing systems
9. Swabs
10. Large binder clip
11. Glass Pyrex dish
12. 2 glass plates
13. Glycerol
14. Oven mitts
15. Polishing tools
16. Water



Tools Required (Steam Washing)

1. Popular steam washer such as Worldent Hot Shot steam cleaner