

# ArgenWAX

## Recommended Setting Parameters



### MATERIAL SELECTION INDICATIONS

Printed Wax  
Milled Wax  
Carvable Wax

**CUT-OFF TIME** Files must be submitted by 3:00 p.m. PST (6:00 p.m. EST)  
Digital Wax units will be received within 2 business days.

**DESIGN** All undercuts should be blocked out prior to scanning or in the design software.  
Anterior bridge connectors should be 3 mm by 2 mm.  
Posterior bridge connectors should be 3 mm by 3 mm.

### SETTING FOR COPING AND CROWNS

	Cement Gap	Extra Cement Gap	Distance to Margin	Wall Thickness	Margin Thickness	Margin Angle
Printed	.02	.05	1.7	.40	.20	90°
Milled	.02	.05	1.7	.40	.15	90°
Carvable	.01	.04	1.7	.45	.15	65°

*Note: Cement gap settings are adjustable. Increase values to loosen fit, decrease values to get a tighter fit.*

**MATERIAL SELECTION GUIDE** We offer single unit copings up to 14-unit bridges in every ArgenWAX product.  
When submitting your files to our website, please ensure to use the appropriate drop down fields.



Send your files to:

**Argen Digital**  
[www.argen.com/link](http://www.argen.com/link)

Files can be uploaded one at a time, or in batches, making the upload process easy!  
Please call Argen Digital for pricing details.

