

GET THE RESULTS YOU WANT, FASTER

Get the same esthetic results faster with the KDF Zircom Speed furnace. Speed cycles have been validated for ArgenZ HT+ and ArgenZ HT+ Multilayer.



105 Minute Cycle

For single units and short span bridges.
Full oven/full tray.

Step	Temperature	Time
1	900°C	15
2	1525°C	20
3	1525°C	45
4	1400°C	2
5	1200°C	3
6	800°C	10
7	350°C	10
Total time:		105 minutes

90 Minute Cycle

For single units only.
Full oven/full tray.

Step	Temperature	Time
1	1000°C	12
2	1510°C	18
3	1510°C	30
4	1400°C	3
5	1200°C	5
6	800°C	12
7	350°C	10
Total time:		90 minutes



argen.com



BD-18864-Ver1-12/21